

CC Shorty

Protocolo Cirugía



Diámetro (Ø) mm
Ø 4.25

Secuencia
quirúrgica
recomendada



Fresa	FRESA INICIAL	DRP200 DRP230	DRP280	DRP310	DRP360	CTK425*
Tope		STC2506	STC2506	STC2506	STC3406	SCTK425



Diámetro (Ø) mm
Ø 4.75

Secuencia
quirúrgica
recomendada



Fresa	FRESA INICIAL	DRP200 DRP230	DRP280	DRP310	DRP360	DRP410	CTK475*
Tope		STC2506	STC2506	STC2506	STC3406	STC3406	SCTK475



Diámetro (Ø) mm
Ø 5.50

Secuencia
quirúrgica
recomendada



Fresa	FRESA INICIAL	DRP200 DRP230	DRP280	DRP310	DRP360	DRP410	DRP485	DRP500	CTK550*
Tope		STC2506	STC2506	STC2506	STC3406	STC3406	STC5006 BAJA DENSIDAD ÓSEA	STC5006 ALTA DENSIDAD ÓSEA	SCTK550