

i•s-Clean tapered KIT


DPD


DDE


Ø2.2


Ø2.8


Ø2.8


Ø3.7


Ø4.1

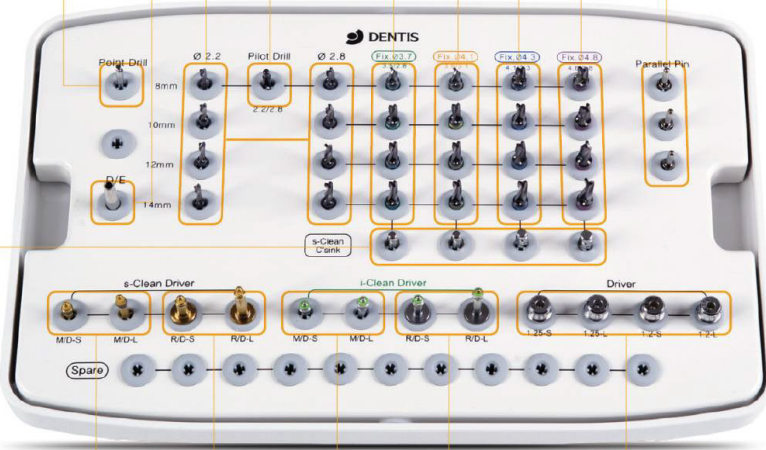

Ø4.3


Ø4.8


DPP

Point Drill	Drill Extension	Straight Twist Drill	Pilot Drill	Twist Drill	Parallel Pin
-------------	-----------------	----------------------	-------------	-------------	--------------


DPDG


Code – DIK


DTR

i•s-Clean Tapered countersink	s-Clean No-mount Driver	s-Clean Ratchet Driver	i-Clean No-mount Driver	i-Clean Ratchet Driver	Hex Driver
-------------------------------	-------------------------	------------------------	-------------------------	------------------------	------------


Ø3.7


Ø4.1


Ø4.3


Ø4.8


DSNDS
DSNDL


DRMDSS
DRMSDL


DINDS
DINDL


DRMDIS
DRMDIL


1.2 Hex
DRHDS12
DRHDL12


1.25 Hex
DRHDS125
DRHDL125

DSTCS37 DSTCS41 DSTCS43 DSTCS48

DSNDS DSNDL

DRMDSS DRMSDL

DINDS DINDL

DRMDIS DRMDIL

1.2 Hex DRHDS12 DRHDL12

1.25 Hex DRHDS125 DRHDL125

i•s-Clean Drilling Sequence T1,T2

Ø 3.7 x 12mm



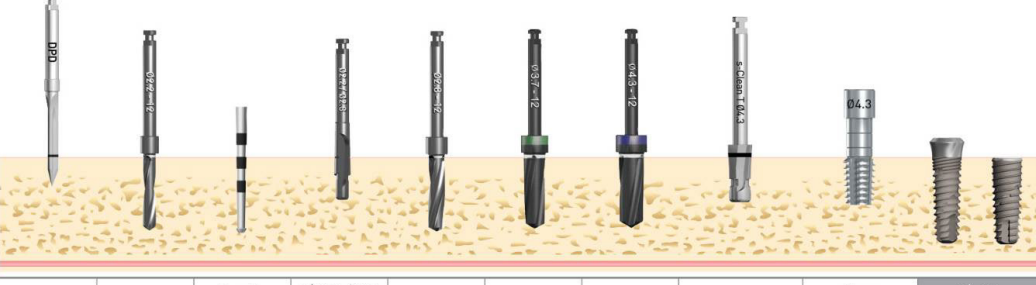
Point drill	Ø 2.2 drill	Depth gauge	Ø 2.2 / 2.8 Pilot drill	Ø 2.8 drill	Ø 3.7 drill	Countersink	Tap (Option)	Ø 3.7 Fixture
-------------	-------------	-------------	-------------------------	-------------	-------------	-------------	--------------	---------------

Ø 4.1 x 12mm



Point drill	Ø 2.2 drill	Depth gauge	Ø 2.2 / 2.8 Pilot drill	Ø 2.8 drill	Ø 3.7 drill	Ø 4.1 drill	Countersink	Tap (Option)	Ø 4.1 Fixture
-------------	-------------	-------------	-------------------------	-------------	-------------	-------------	-------------	--------------	---------------

Ø 4.3 x 12mm



Point drill	Ø 2.2 drill	Depth gauge	Ø 2.2 / 2.8 Pilot drill	Ø 2.8 drill	Ø 3.7 drill	Ø 4.3 drill	Countersink	Tap (Option)	Ø 4.3 Fixture
-------------	-------------	-------------	-------------------------	-------------	-------------	-------------	-------------	--------------	---------------

Ø 4.8 x 12mm



Point drill	Ø 2.2 drill	Depth gauge	Ø 2.2 / 2.8 Pilot drill	Ø 2.8 drill	Ø 3.7 drill	Ø 4.3 drill	Ø 4.8 drill	Countersink	Tap (Option)	Ø 4.8 Fixture
-------------	-------------	-------------	-------------------------	-------------	-------------	-------------	-------------	-------------	--------------	---------------